



CONTENTS

PRODUCT NO.

PACKAGE

ALP KIT - C	C174-0C	
ALP SAMPLE DILUENT	C174-05	6 x 22.5 mL
ALP ACTIVATOR	C174-06	6 x 2.5mL
ALP KIT - D	C174-0D	
ALP SAMPLE DILUENT	C174-07	6 x 45 mL
ALP ACTIVATOR	C174-08	6 x 5.0 mL

WORKING REAGENT PREPARATION

Transfer the contents of one ALP Activator Reagent vial (C174-06 or C174-08) into appropriate bottle of ALP Sample Diluent Reagent (C174-05 or C174-07). If necessary, rinse Activator vial, with combined Sample Diluent Reagent. Mix the combined reagent in Sample Diluent Reagent vial by inverting bottle several times.

*(If used as a two reagent system use appropriate instrument application.)*

DIRECTIONS FOR USE

1. When prepared as directed, the working reagent is ready for use.
2. The ALP Working Reagent is stable for 15 days at 2-8° C and capped while not in use.

**FOR TECHNICAL ASSISTANCE PLEASE CALL:**

**203-262-0330**

**OR**

**SUBMIT A CONTACT FORM AT**  
**[www.catacheminc.com](http://www.catacheminc.com)**