



Vet Spec
ALKALINE PHOSPHATASE (ALP) REAGENTS
C174-0C; C174-0D

<u>CONTENTS</u>	<u>PRODUCT NO.</u>	<u>PACKAGE</u>
ALP KIT - C	C174-0C	
ALP SAMPLE DILUENT	C174-05	6 x 22.5 mL
ALP ACTIVATOR	C174-06	6 x 2.5mL
ALP KIT - D	C174-0D	
ALP SAMPLE DILUENT	C174-07	6 x 45 mL
ALP ACTIVATOR	C174-08	6 x 5.0 mL

WORKING REAGENT PREPARATION

Transfer the contents of one ALP Activator Reagent vial (C174-06 or C174-08) into appropriate bottle of ALP Sample Diluent Reagent (C174-05 or C174-07). If necessary, rinse Activator vial, with combined Sample Diluent Reagent. Mix the combined reagent in Sample Diluent Reagent vial by inverting bottle several times.

(If used as a two reagent system use appropriate instrument application.)

DIRECTIONS FOR USE

1. When prepared as directed, the working reagent is ready for use.
2. The ALP Working Reagent is stable for 15 days at 2-8° C and capped while not in use.

FOR TECHNICAL ASSISTANCE PLEASE CALL:

203-262-0330

OR

SUBMIT A CONTACT FORM AT
www.catacheminc.com