



ALKALINE PHOSPHATASE (ALP) REAGENT KIT  
V174-14

**CONTENTS**

**PRODUCT NO.**

**PACKAGING**

ALP REAGENT KIT	V174-14	
ALP SAMPLE DILUENT (R1)	V174-03	3 x 13 mL
ALP ACTIVATOR (R2)	V174-04	3 x 3.5 mL

**REAGENT PREPARATION**

These reagents are packaged ready for use. No preparation is required.

**REAGENT STORAGE AND STABILITY**

Store unopened reagents at 2-8° C (refrigerated). When stored as directed, the reagents are stable until the expiration date stated on the label.

**DIRECTIONS FOR USE**

The Catachem Alkaline Phosphatase test method requires two reagents, R1 and R2. Use the ALP Sample Diluent as R1 and the ALP Activator as R2.

After opening, the reagents are stable on board the analyzer at 2-8°C (refrigerated) for 30 days. Cap reagents tightly when not in use. NOT FOR USE IN UNPROFESSIONAL SETTINGS.

**FOR TECHNICAL ASSISTANCE PLEASE CALL:**

**(203) 262-0330**

**OR EMAIL:**

**[catachem@catacheminc.com](mailto:catachem@catacheminc.com)**

**[www.catacheminc.com](http://www.catacheminc.com)**