



ALANINE AMINOTRANSFERASE (SGPT) REAGENT
V164-0A

<u>CONTENTS</u>	<u>PRODUCT NO.</u>	<u>PACKAGING</u>
ALT KIT A	V164-0A	
ALT SUBSTRATE (R1)	V164-07	3 x 13.0 mL
ALT ACTIVATOR (R2)	V164-08	3 x 2.5 mL

REAGENT PREPARATION

The ALT SUBSTRATE (R1) (V164-07) reagent and the ALT ACTIVATOR (R2) (V164-08) reagent are packaged ready for use. No preparation is required.

REAGENT STORAGE AND STABILITY

Store unopened reagents at 2-8°C (refrigerated). When stored as directed, the reagents are stable until the expiry date stated on the label.

DIRECTIONS FOR USE

The Catachem ALT Reagents, after opening, are stable on board the analyzer for 30 days at 2-8°C (refrigerated). Cap reagents tightly when not in use. NOT FOR USE IN UNPROFESSIONAL SETTINGS.

FOR TECHNICAL ASSISTANCE PLEASE CALL:

(203) 262-0330

OR EMAIL:

catachem@catacheminc.com

www.catacheminc.com