



PRODUCT DESCRIPTION:

PRODUCT NO.

PACKAGING

BROMIDE REAGENT KIT

V424-0C

Bromide Sample Diluent (R1)

V424-03

2 x 13 mL

Bromide Color Reagent (R2)

V424-04

2 x 6.5 mL

Bromide Calibrator

V424-20

2 x 2.5 mL

Bromide Control Level 1

V424-21

2 x 2.5 mL

Bromide Control Level 2

V424-22

2 x 2.5 mL

REAGENT PREPARATION:

The Catachem Bromide reagents are packaged ready for use. No preparation is required for either the Sample Diluent (R1) or the Color Reagent (R2). The Catachem Bromide Calibrator and Control Levels 1 & 2 are packaged ready to use and require no preparation.

REAGENT STORAGE AND STABILITY:

Store unopened reagents at 2-8°C (refrigerated). When stored as directed, the reagents are stable until the expiration dates stated on the labels. Store unopened calibrator and controls at 2-8°C. When stored as directed, the calibrator and controls are stable until the expiration dates stated on the labels. The reagents, calibrator and controls have been tested to reflect shipping conditions and are stable for the lifespan of the product if frozen up to 5 times or reaching temperatures up to 40°C for up to one week.

DIRECTIONS FOR USE:

The Catachem, Inc. Bromide test method requires two reagents, R1 and R2. Use the Bromide Sample Diluent as R1 and the Bromide Color Reagent as R2. Upon opening, the reagents, calibrator and controls are stable for at least 60 days at 2-8°C. Keep reagents, calibrators and controls tightly capped when not in use. **NOT FOR USE IN UNPROFESSIONAL SETTINGS.**

FOR TECHNICAL ASSISTANCE PLEASE CONTACT:

Telephone: 203-262-0330

Or use the contact form at

www.catacheminc.com