



*Vet Spec* TRIGLYCERIDES REAGENT  
C115-0A, C115-0B, V115-12

| <u>CONTENTS</u>           | <u>PRODUCT NO.</u> | <u>PACKAGING</u> |
|---------------------------|--------------------|------------------|
| TRIGLYCERIDES GPO REAGENT | C115-0A            |                  |
| TRIGLYCERIDES REAGENT     | C115-01            | 6 x 0.372g       |
| TRIGLYCERIDES DILUENT     | C115-02            | 6 x 25 mL        |
| TRIGLYCERIDES GPO REAGENT | C115-0B            |                  |
| TRIGLYCERIDES REAGENT     | C115-03            | 6 x 0.745g       |
| TRIGLYCERIDES DILUENT     | C115-04            | 6 x 50 mL        |
| TRIGLYCERIDES GPO REAGENT | V115-12            |                  |
| TRIGLYCERIDES REAGENT     | V115-13            | 6 x 0.194g       |
| TRIGLYCERIDES DILUENT     | V115-14            | 6 x 13 mL        |

**REAGENT PREPARATION**

Add contents of one bottle of diluent into one bottle of powder. Mix gently until fully dissolved.

**REAGENT STORAGE AND STABILITY**

After reconstitution the working reagent is stable in liquid form for at least 14 days when stored refrigerated (2-8C).

**DIRECTIONS FOR USE**

After reconstitution the reagent is ready for use on any automated or manual analyzer as a single reagent. NOT FOR USE IN UNPROFESSIONAL SETTINGS.

**FOR TECHNICAL ASSISTANCE PLEASE CONTACT Catachem at  
[catachem@catacheminc.com](mailto:catachem@catacheminc.com); Tel 203-262-0330**

[www.catacheminc.com](http://www.catacheminc.com)