



**PYRUVATE REAGENT KIT
C750-0A**

<u>PRODUCT NO.</u>	<u>CONTENTS</u>	<u>PACKAGING</u>
PYRUVATE REAGENT KIT	C750-0A	
PYRUVATE REAGENT	C750-01	6 x 25 mL

REAGENT PREPARATION

This reagent is a dry powder reagent. Bring one bottle of Pyruvate Reagent, Product Number: C750-01 to 18-26°C (room temperature). Pour 25 mL of deionized water into the bottle. Mix contents by inversion for approximately 5-10 minutes until all of the reagent is in solution.

Store the reconstituted working reagent at 2-8°C (refrigerated).

REAGENT STORAGE AND STABILITY

Store the dry powder reagent at 2-8°C (refrigerated). When stored as directed, the reagent is stable until the expiration date stated on the label.

DIRECTIONS FOR USE

The Catachem, Inc. Pyruvate Working Reagent after reconstitution is stable on board the analyzer at 2-8°C (refrigerated) for at least 7 days. Cap single working reagent tightly when not in use. NOT FOR USE IN UNPROFESSIONAL SETTINGS.

FOR TECHNICAL ASSISTANCE PLEASE CALL:

(203) 262-0330

OR EMAIL

catachem@catacheminc.com

www.catacheminc.com